

JARIT®



JARIT® ARTICULATING FAN RETRACTOR

Simple • Reusable • Easy to Clean

 **INTEGRA™**
Surgical




- 10MM DIAMETER, 380MM WORKING LENGTH
- FIVE BLADES (68MM LENGTH), SPREAD TO A FULL 60 DEGREES
- FAN ARTICULATES TO 80 DEGREES MAXIMUM TILT

ASSEMBLY



STEP 1
Screw the fan assembly into the shaft.

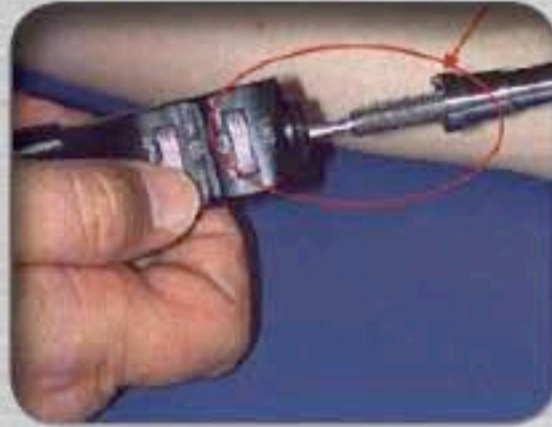

STEP 2
Insert the shaft into the handle while aligning the guide pin (see arrow) with the Index hole in the handle.



STEP 3
Turn the "LIFT" wheel towards "UP" until the thread is fully engaged.



STEP 4
Slide the cap over the shaft and screw it on to the handle.

COMPLETELY ASSEMBLED


DISASSEMBLY



STEP 1
Unscrew the cap and remove by sliding over the shaft.



STEP 2
Turn the "LIFT" wheel towards "DOWN" as the threaded end of the outer shaft is removed.

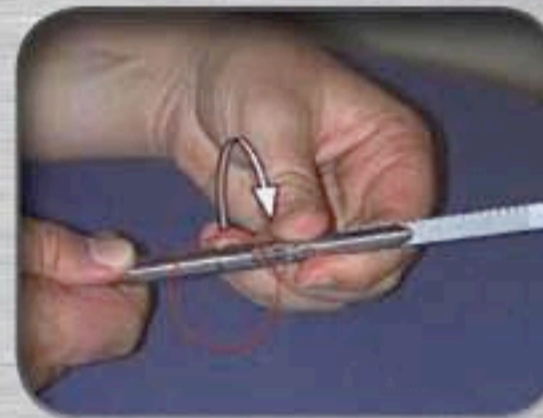

STEP 3
Unscrew the fan assembly and remove from the outer shaft.



COMPLETELY DISASSEMBLED

ORDERING INFORMATION

Part No.	Description	Working Length	Diameter
600-540	JARIT Articulating Fan Retractor	380mm	10mm



www.Integra-Surgical.com

Jarit is a registered trademark of J. Jamner Surgical Instruments, Inc., a subsidiary of Integra LifeSciences Corporation. The Integra wave logo and Integra Surgical are trademarks of Integra LifeSciences Corporation. Copyright © 2009 Integra LifeSciences Corporation. All rights reserved.

THE INSTRUMENT PEOPLE

LUXTEC | OMNITRACT | JARIT | PADGETT | REDMOND